ACCU-CHEK® Safe-T-Pro Uno

EN Sterile, single-use lancing device

Read these instructions for use before you obtain blood with this sterile, single-use lancing device.

These instructions for use feature the following 2 symbols:

⚠ This symbol indicates a possible risk of injury or of damage to your health.

ⓘ This symbol draws your attention to important information.

Intended Use

The sterile, single-use lancing device is intended for obtaining capillary blood from the fingertip or, if the patient is a child under 1 year, from the heel.

Components of the single-use lancing device (see illustration ⓘ)

- ① Description
- ② Release button
- ③ Housing
- ④ Sterility cap of the needle

The special cut and small diameter of the needle (Ø 0.36 mm, 28 G) allow you to use this lancing device to obtain blood from your patients in a way that is virtually pain free.

The sterility cap keeps the needle sterile.

Preparing to obtain blood

② Children younger than 1 year

With children younger than 1 year of age, the capillary blood must be obtained from the heel. Obtaining blood must occur from the lateral areas of the sole (see black areas in illustration ⓘ).

The puncture must occur vertically to the sole (see arrows in the enlargement). While obtaining blood, the child should be held with its foot vertically facing the floor. A site which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before obtaining blood, you can carefully warm the heel, for example by wrapping it in a warm, moist cloth of a maximum 42 °C.

Wash the child's heel with warm water and soap and rinse thoroughly. Disinfect the heel with an alcohol wipe. Afterwards, let the heel dry completely.

③ Children as of 1 year and adults

We recommend obtaining capillary blood from the side of the fingertip, as this puncture site registers the least amount of pain. Any finger can be used for the puncture (see black areas in illustration ⓘ).

Wash the patient's hands with warm water and soap and rinse thoroughly. Afterwards, let the hands dry completely. This ensures that the puncture site is clean and increases blood flow. If you are using alcohol wipes in addition, the finger must also be allowed to dry completely.

Obtaining blood

④ Removing the sterility cap

► Twist the sterility cap one quarter turn and remove it (see illustration ⓘ).

⚠ Do not use the single-use lancing device if the sterility cap has been previously removed.

⑤ + ⑥ Obtaining blood

► Hold the single-use lancing device between the index finger, middle finger and thumb (see illustration ⓘ).

- Press the single-use lancing device firmly against the chosen puncture site.
- Using your thumb, press the release button down completely (see illustration ⓘ).
- Remove the single-use lancing device from the puncture site.
- Wait for a blood drop to emerge from the puncture site.
- Gently massage the finger towards the fingertip or exert slight pressure on the heel to encourage a blood drop to form.

⚠ For some diagnostic tests, the fingertip/heel must not be vigorously massaged or compressed, as doing so may produce incorrect test results.

ⓘ Always discard used single-use lancing devices in accordance with your facility's disposal procedures.

Last update

2014-12

Customer Support and Service Centre

Roche Diabetes Care Limited

Charles Avenue, Burgess Hill

West Sussex, RH15 9RY, United Kingdom

Accu-Chek Customer Caroline¹⁾

UK Freephone number: 0800 701 000

ROI Freephone number: 1 800 709 600

¹⁾ calls may be recorded for training purposes

www.accu-check.co.uk

www.accu-check.ie

Vivian Corporation

Sanitas Building

Tower Street, Msida MSD1824, Malta

Phone: 80073102

The sterile, single-use lancing device is intended for obtaining capillary blood from the fingertip or, if the patient is a child under 1 year, from the heel.

ⓘ Components of the single-use lancing device (see illustration ⓘ)

- ① Description
- ② Release button
- ③ Housing
- ④ Sterility cap of the needle

The special cut and small diameter of the needle (Ø 0.36 mm, 28 G) allow you to use this lancing device to obtain blood from your patients in a way that is virtually pain free.

The sterility cap keeps the needle sterile.

Preparing to obtain blood

② Children younger than 1 year

With children younger than 1 year of age, the capillary blood must be obtained from the heel. Obtaining blood must occur from the lateral areas of the sole (see black areas in illustration ⓘ).

The puncture must occur vertically to the sole (see arrows in the enlargement). While obtaining blood, the child should be held with its foot vertically facing the floor. A site which has already been punctured must not be punctured a second time.

To improve blood flow at the heel before obtaining blood, you can carefully warm the heel, for example by wrapping it in a warm, moist cloth of a maximum 42 °C.

Wash the child's heel with warm water and soap and rinse thoroughly. Disinfect the heel with an alcohol wipe. Afterwards, let the heel dry completely.

③ Children as of 1 year and adults

We recommend obtaining capillary blood from the side of the fingertip, as this puncture site registers the least amount of pain. Any finger can be used for the puncture (see black areas in illustration ⓘ).

Wash the patient's hands with warm water and soap and rinse thoroughly. Afterwards, let the hands dry completely. This ensures that the puncture site is clean and increases blood flow. If you are using alcohol wipes in addition, the finger must also be allowed to dry completely.

Obtaining blood

④ Removing the sterility cap

► Twist the sterility cap one quarter turn and remove it (see illustration ⓘ).

⚠ Do not use the single-use lancing device if the sterility cap has been previously removed.

⑤ + ⑥ Obtaining blood

► Hold the single-use lancing device between the index finger, middle finger and thumb (see illustration ⓘ).

The sterility cap keeps the needle sterile.

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